

# Bio Cart — 360L

GREY BASE AND LID | MAX WEIGHT: 58KG



MAKING HEALTHCARE SAFER  
SOINS DE SANTÉ PLUS SÉCURITAIRES



1

Open lid of reusable biomedical waste container.



2

If using a liner, install by folding over top edges. Fill with biomedical waste.



3

If not using a liner, fill with biomedical waste that is in tied/sealed bags only.



4

Insert filled sharp containers that are sealed and securely locked only.



5

Seal all liners by twisting and hand-tying into a single knot.



6

Close lid and securely store in waste collection area. **No liners should be hanging over the edge once the lid has been closed.**



7

Place white Daniels barcode label on the upper left hand side of the front of the Bio Cart.



8

Contact your local Daniels service location for a pick-up of your container.