


Cardboard Box

BROWN BOX – YELLOW BAG



MAKING HEALTHCARE SAFER
SOINS DE SANTÉ PLUS SÉCURITAIRES

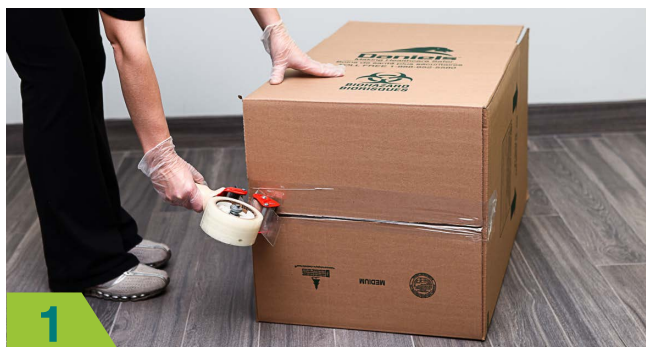

1

Turn over and seal bottom flaps with tape



2

Place liner into box and fold over the edges. Fill box with Biomedical waste. **No loose sharps are permitted outside of sharp collectors.**



3

Close lid by tapping the flaps down. Securely store in waste collection. **Liners should not be showing once the flaps have been taped.**



4

Seal all liners by twisting and hand-tying into a single knot.



5

Where applicable, place colour coded waste ID label beside white Daniels barcode label.



6

Contact your local Daniels service location for a pick-up of your container.

Small Cardboard Box

Dimensions: 14" x 14" x 21"
Max Weight: 16kg

Big Cardboard Box

Dimensions: 18" x 18" x 24"
Max Weight: 23kg

CAN181009

WWW.DANIELSHEALTH.CA | WWW.ECOSHIP.CA