

# Ecoship Box

PER SIZE



MAKING HEALTHCARE SAFER  
SOINS DE SANTÉ PLUS SÉCURITAIRES



## CONFIGURATIONS

### Medium Box

- 2 x S14 Collectors or
- 1 x S14 Collector & 1 x S22 Collector or
- 2 x S22 Collectors

### Large Box


- 4 x S14 Collectors or
- 2 x S32 Collectors or
- 1 x S32 Collector & 1 x S22 Collector or
- 1 x S32 Collectors & 2 x S14 Collectors



1 Open Ecoship Box and distribute sharp collectors throughout facility where required. **Ensure paperwork and empty box are stored in a safe place.**



2 Ensure the flaps on the inside of the Ecoship box are open towards the side.

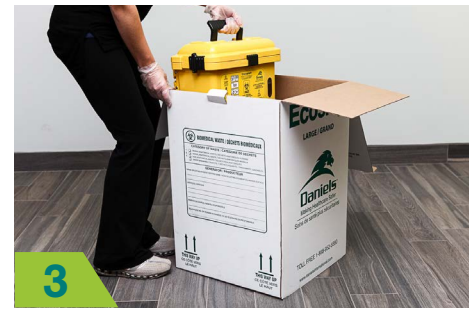

3 Place locked sharp collectors in standing position inside the Ecoship Box.



4 To close the box, fold side flaps over closed sharp collectors, fold lid and tuck into the front of box. **Ensure security tab is fully inserted into lid.**



5 If applicable, place return bill of lading inside the provided labelope. If you were not provided a return bill of lading, contact Daniels to have one sent to you via email or fax.



6 Ensure the inner flaps are upright when placing locked sharp containers inside the box. This will ensure the box stays secure through transport.