

Pail Packaging



MAKING HEALTHCARE SAFER
SOINS DE SANTÉ PLUS SÉCURITAIRES



1

Ensure the pail is only filled with approved waste.
Place lid flat across the opening of the pail.
Ensure the pail is not overfilled.



2

Do not remove the tamper proof seal.



3

Close lid securely. Please ensure there are no visible gaps in the gasket of the lid. Continue to apply pressure until the lid is fully closed.



4

Affix white Daniels barcode label to the side of the pail.



5

Where applicable, place colour coded waste ID label beside white Daniels barcode label.



6

Contact your local Daniels service location for a pick-up of your pail.