


How to properly package and dispose the Yellow Pail



1 Remove pail from the Ecoship box. **Ensure box is stored in a safe place (you will need it for return shipment)**

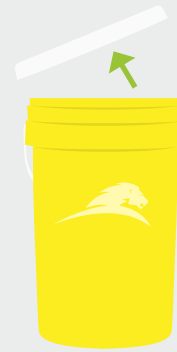

2 Remove the lid from the top of the pail. **Ensure the lid is stored in a safe place (you will need it for return shipment).**



3 Dispose of waste in the pail



4 When the pail is full, replace the lid and ensure it is securely closed.



5 Place the pail back in the Ecoship box

NOTES ON SHIPMENT

- Purolator arranges for all shipment back to Daniels Health within all provinces with **exception of Quebec and British Columbia** – this is completed within their network.
- Quebec and British Columbia will be delivered to a central depot and Daniels will pick up bulk shipments directly.
- Generic pails and boxes cannot be used. **Only supplied pails and boxes provided by Daniels Health are acceptable**