

How To Use Your Secure a Drug



UNDER DIRECT OBSERVATION OF A WITNESS:

- 

1 Remove medication from packaging: vial, bubble pack or foil
- 

2 Load unused or unwanted controlled medications into the Secure a Drug container
- 

3 Gently agitate container so the liquid slurry pours over the medications
- 

4 When the container is full, tighten cap securely to the left aligning the triangle shape on the lid to the "locked" position. Safely dispose of locked container into a non-hazardous or hazardous waste bin.

CORRECT DISPOSAL

Full, used Secure a Drug containers will be returned to Pharmacy for storage until the contracted vendor collects and disposes of the waste.

If Hazardous controlled substances are disposed of into Secure a Drug, the full containers MUST be placed into a Hazardous container.

SADC211005

PLEASE NOTE

Empty vials and ampoules must go into the sharps container. DO NOT place sharps or syringes into Secure a Drug containers. Never dispose of Secure a Drug containers in general trash or biomedical bins.

866 994 5205

WWW.SECUREADRUG.CA



SECURE
a DRUG