



Sharpsmart

REGULAR | INSTRUCTIONS FOR USE

OPEN

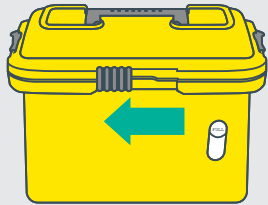


- 1 Slide the front lock to the left.

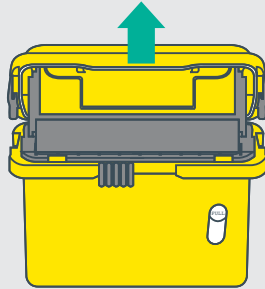


- 2 Lift lid until it locks open.

SYRINGES

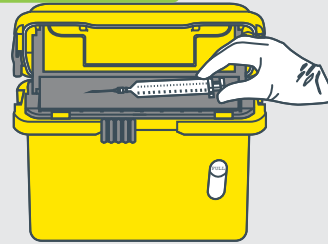


- 1 Drop syringes horizontally onto tray.

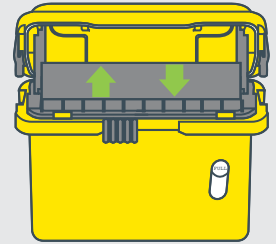


- 2 Tray will empty itself.

LIGHT SHARPS

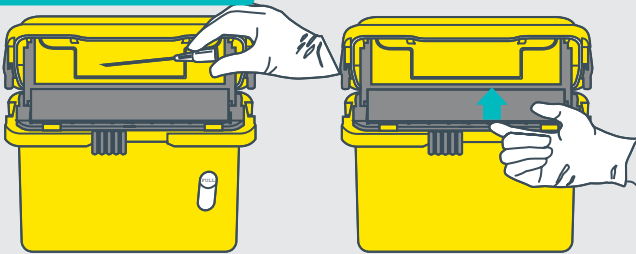

- 1 Drop sharp horizontally onto tray.
- 2 Manually lift the tray to empty.

FULL

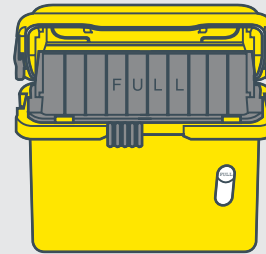

When the tray remains vertical, lock & replace the collector.

CLOSE




- 1 Press both side supports inward, close lid.



- 2 Slide the front lock fully to the right.

LOCK

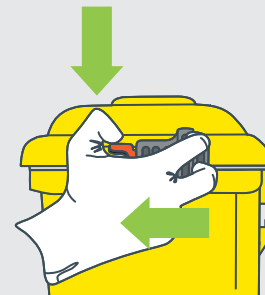

- 1 Depress red tabs.

- 2 Slide locks completely forward.

PLEASE NOTE
Side locks are permanent!

Remember

Never put your hands into a sharps collector.
Always close the collector before moving or removing container.
Do not deface or write on the collector.
Always dispose of sharps and sharps safety devices into sharps collectors.