

Cardboard Box

BROWN BOX WITH RED STRIPE AND RED LINER



MAKING HEALTHCARE SAFER
SOINS DE SANTÉ PLUS SÉCURITAIRES



1

Turn over and seal bottom flaps with packing tape



2

Place Red liner into box. Fill box with Incinerate-only waste. **No loose sharps are permitted outside of sharp containers.**



3

Seal all liners by twisting and hand-tying into a single knot.



4

Close lid by taping the flaps down. Securely store in waste collection area. **Liners should not be showing once the flaps have been taped.**



5

Where applicable, place 4*4 waste place identification Label next to Daniels Barcode Label. **For example: Cytotoxic sticker, or Pharmaceutical Sticker.**



6

Contact your local Daniels service location for a pick-up of your container.



WWW.DANIELSHEALTH.CA

Cardboard Box

Dimensions: 24"x13"x19"

Max Weight: 16KG / 35LB