

# Cardboard Box

BROWN BOX WITH YELLOW STRIPE AND YELLOW LINER



MAKING HEALTHCARE SAFER  
SOINS DE SANTÉ PLUS SÉCURITAIRES



1

Turn over and seal bottom flaps with packaging tape



2

Place Yellow liner into box. Fill box with Biomedical waste. **No loose sharps are permitted outside of sharps containers.**



3

Seal all liners by twisting and hand-tying into a single knot.




4

Close lid by taping the flaps down. Securely store in waste collection area. **Liners should not be showing once the flaps have been taped.**



5

Where applicable, place Daniels Barcode label in designated printed area.

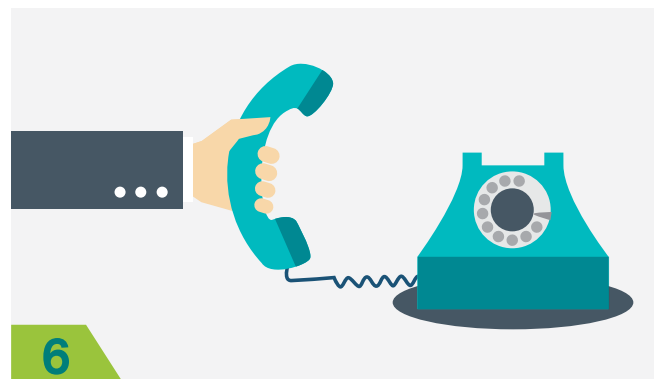

6

Contact your local Daniels service location for a pick-up of your container.

## Cardboard Box

Dimensions: 24"x13"x19"

Max Weight: 16KG / 35LB



WWW.DANIELSHEALTH.CA

CAN181009